


MONITOR INSTALLATION

Losen the set screw at the top bracket

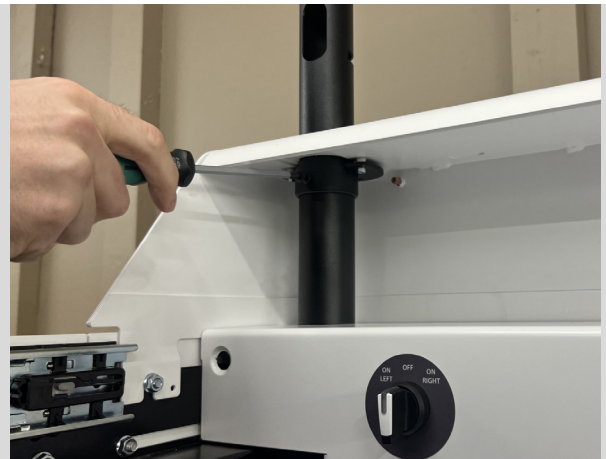

Insert the pole

> Insert the pole into the fitting and ensure you guide it carefully into the proper position



Tighten the bolt in the bottom bracket

> Ensure the screw goes into the slot, locking the movement



Fasten the set screw at the top bracket



Slide scanner arm over pole



Prepare the cables

> Pull necessary cables through before mounting the monitor arm



Secure the scanner and monitor arm

> Fasten top set screw for positioning and tighten the bottom set screw to control the movement



The pole and monitor arm are now ready for use!

